

Chart Type:

Select Red/Green, Fixation, Fourdot, etc.

Control:

Show all, Blank all, and darken background

Choose Symbol Type:

- Letters
- Numbers
- Symbols
- Rolling E's
- Rolling C's

Select Acuity:

Use menu to select an acuity or a set of 3 acuity rows to display

Symbol	Type	Control	Acuity
E			20/400.0
C L			20/200.0
O D P			20/150.0
V F L N			20/100.0
T E D F O			20/80.0
P N T Z F			20/60.0
V C E T L			20/50.0
P V T			20/40.0
D F N L			20/30.0
V O F C			20/25.0
P E V N T			20/20.0
F D C Z N			20/15.0
V T D L C			20/10.0

Toggle Row:

Turn a row off and on.

Single Letter:

Click on any letter to show only one letter.

One Row Only:

Display this row only

RANDOMIZE

BLANK

Randomize:

Randomize the chart with one click.

One Click Blank:

Darken the chart to reduce illumination in the exam room. Return to chart by clicking again.

© 2006, 2005 DigitalEyeChart.com

Hot Key--Keyboard Control

Switch Between Charts

- “s” standard multi-line chart
- “r” red/green chart
- “v” variable low vision chart
- “t” contrast sensitivity chart
- “d” four dot test
- “f” fixation dot
- “g” animatd fixation dot
- “a” astigmatism chart
- “w” slide show
- “b” blank/unblank (darken screen)

Change Optotype

- “l” letters
- “n” numbers
- “p” pictures
- “e” tumbling E’s
- “c” rolling C’s
- “space bar” randomize current type

All Multi-line chart control

- “Home” show all rows
- “End” clear all rows
- Up/Down Arrows -- move to next/previous row
- “m” mask selected items

Contrast Sensitivity

- “PgUp/PgDown” change mean row by 5% +/-

Low Vision Single Letter

- “PgUp/PgDown” change acuity double/half
- Up/Down Arrows -- change acuity level by 10%



An affordable digital eye chart solution.

www.DigitalEyeChart.com

(208) 520 4292

sales@DigitalEyeChart.com

© Copyright 2006-2007 DigitalEyeChart.com

All rights reserved